



Quick Start Guide: AI Creations Space

Guide for Professionals


Getting Started

Before you start

Make sure your computer is connected to the internet and your headset is turned on, charged and connected to the wifi.

Login


- Go to: platform.xr.health
- Use the login credentials you received in the email "**Access your XRHealth account**"
- Select the Space you will be working with for the session

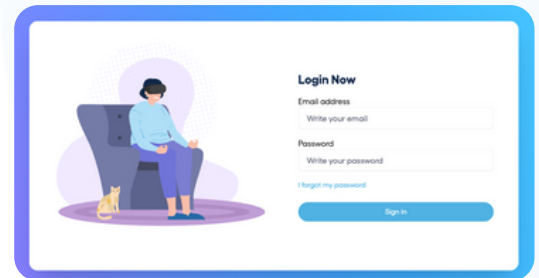

Homepage

Here you will find:

- **Spaces:** The different Spaces available. The colored Spaces are included in your plan.
- **My Favorites:** Environments you've marked as favorites
- **Recent Environments:** Your most recently used environments
- **Popular environments:** The most used environments across XRHealth

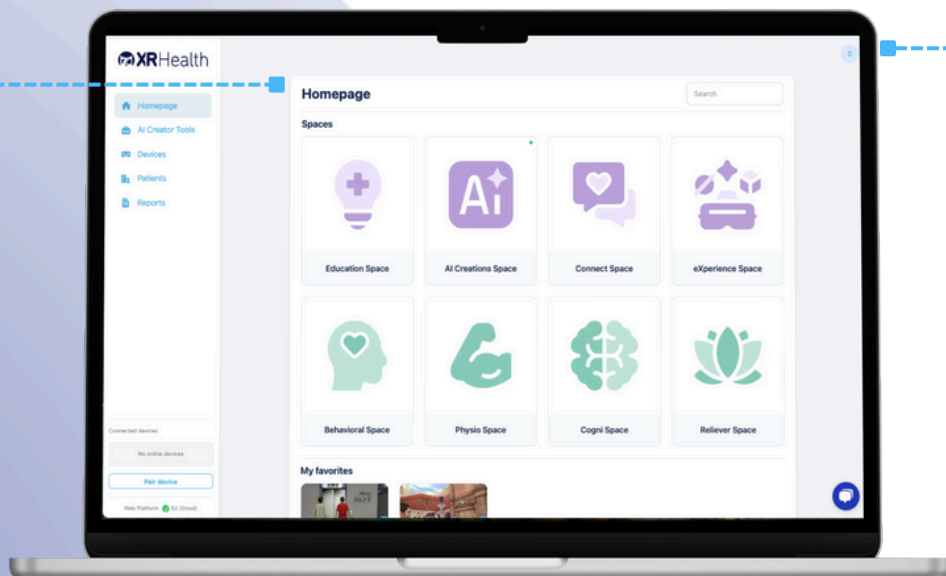

Account Settings

Click on your initials in the top right corner to:

- Update your personal details, language settings, or password
- Note:** To change your email address, contact Customer Success




Quick Start Guide: AI Creations Space

Guide for Professionals

Creating a Virtual Environment


The AI Generative for VR feature allows healthcare professionals to create personalized, immersive therapy environments using just a text prompt. This tool is ideal for tailoring therapy sessions in areas such as mental health, physical rehabilitation, and cognitive training.

- 1 Open the AI Virtual Environment Generator in the **"AI Creator tools"**

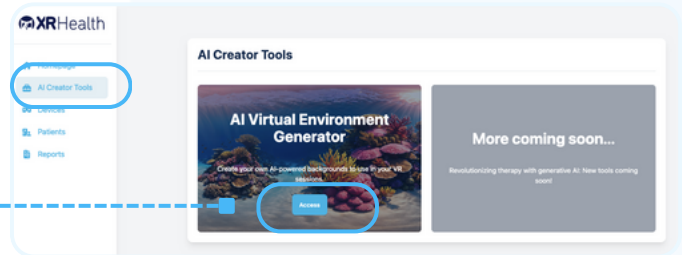

- 2 Click on **"Access"** on the AI Virtual Environment Generator

- 3 Enter a description of the environment you want to create
For example, "magical forest with glowing trees and butterflies"

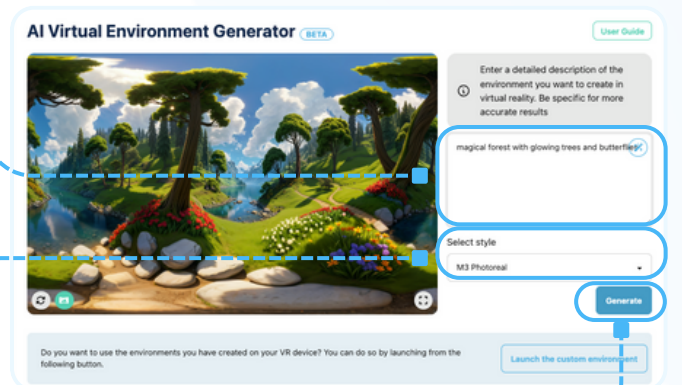


- 4 Select a visual style, such as "M3 photoreal" for a calming effect

- 5 Click **"Generate"**

- 5 If needed, adjust the prompt and regenerate the environment

Note: Environment generation may take some time

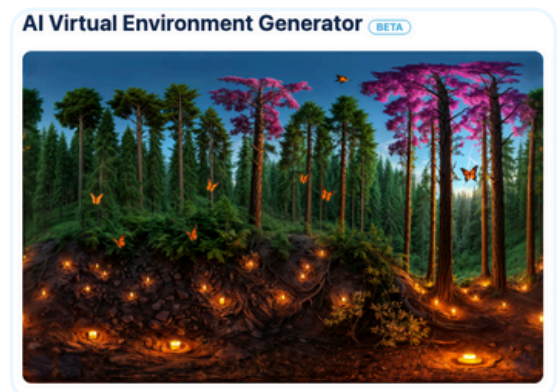



Quick Start Guide: AI Creations Space


Guide for Professionals

Launching Environments

1. Creating a Patient Record

- 1 Click on the **"Patients"** tab
- 2 Click the **"New Patient"** button and enter the patient's email address
 - o The patient will receive a notification email of the account creation

Note: XRHealth complies with GDPR and HIPAA for storing and encrypting patient records.

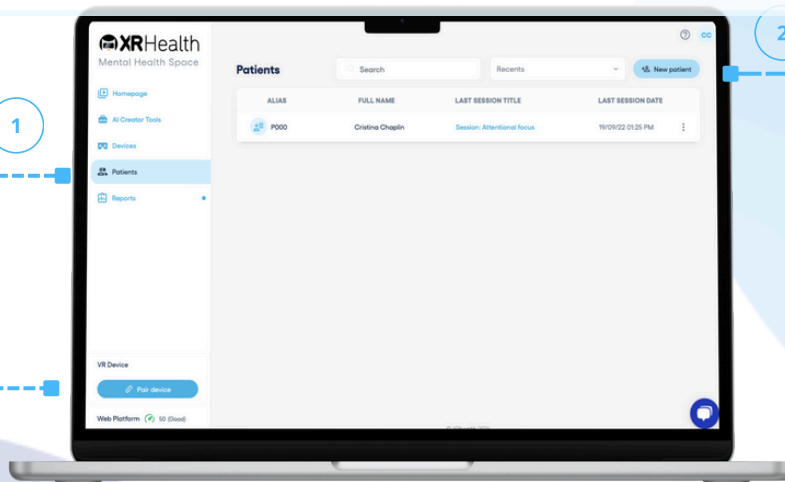




2. Pairing a Headset

In VR headset:

- 1 Select **"AI Creations Space"**
- 2 A 4-character pairing code will appear on the device

In the web Platform:

- 3 Click the **"Pair Device"** button at the bottom-left corner of the platform
- 4 Enter the code in the platform and click **"Pair"**

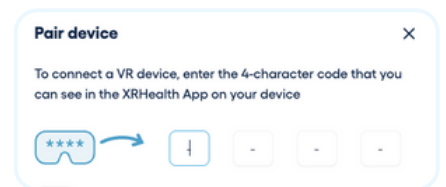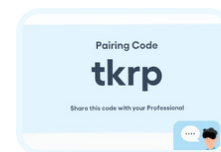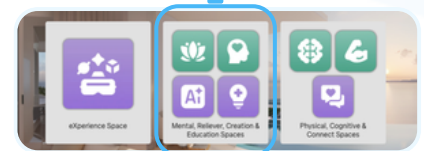




Quick Start Guide: AI Creations Space

Guide for Professionals


3. Starting a VR Session

- 1 Pair your VR headset using the quick pairing code
Note: Pairing is automatic after the first time


- 2 In the Homepage, select the **AI Creations Space**
Note: The colored Spaces are those included in your plan

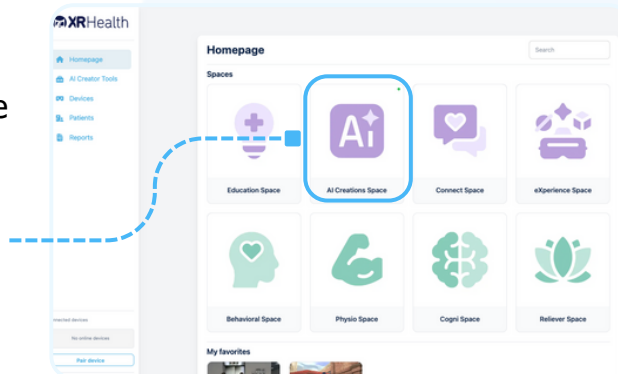


- 3 Click on **Launch Environment**

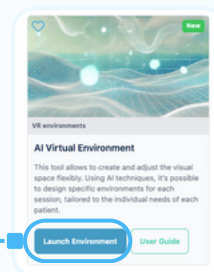

- 4 Choose your patient, complete the session details and click **Start Session** to begin

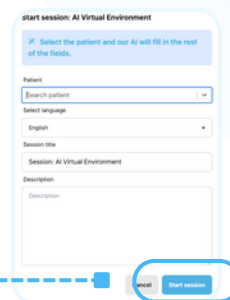

- 5 Select the generated environment matching your session goals and click **Send to device**

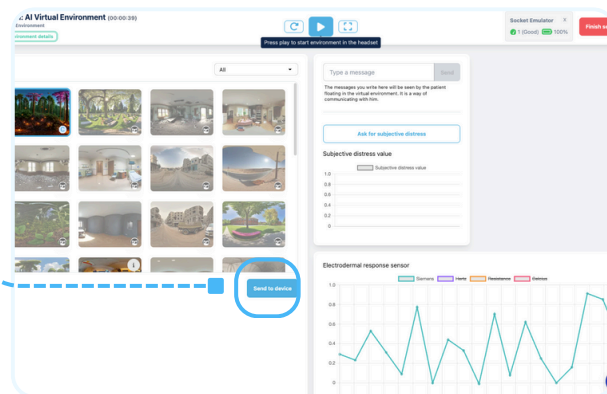

- 6 Click **Play** to start the session in the patient's headset
Note: The environment will play only in the headset, not on the platform.





Quick Start Guide: AI Creations Space

Guide for Professionals

4. Session Report

Review the end-of-session report

- 1 Select the **Reports** section
- 2 Search and select the report
- 3 Access the session report and view data for further insights

